

Hydration-Hydrogen Power

Skin Beauty Machine

Hydration-Hydrogen Power Skin Beauty Machine Introduction

The world's first saturated hydrogen water detoxifying beauty instrument for removing reactive oxygen species. Titanium rapid hydrogen production technology can instantly produce hydrogen-rich water with a concentration of up to 1480 ppb. After skin penetration, it rapidly removes free radicals inside and outside cells, repairs it from the most basic cellular level, improves skin health, and counteracts skin aging caused by free radicals. With the following features: Blemish, acne, whitening, fine lines, repair damage, improve skin immunity, delay skin aging.



Hydration-Hydrogen Power Skin Beauty Machine

Work Therapy

What is a free radical?


1. Free radicals in the body: It is a kind of harmful substance that the human body inevitably produces in the metabolism
2. Environmental free radicals: sunlight, air pollution, smoking, and pesticides all cause the body to produce more reactive oxygen species.

All of this attack the cell membrane, cell nucleus, and DNA. Causes human inflammation, autoimmune diseases, radiation damage, aging, skin diseases.

How to fight free radicals ?

The damage of reactive oxygen radicals to the human body is actually an oxidation process. To reduce the damage of free radicals, we must start with antioxidants. There are also two ways to reduce the harm of free radicals. First, use an endogenous free radical scavenging system to remove excess free radicals in the body. Second, an exogenous antioxidant, a free radical scavenger, blocks free radicals from invading the human body.

The hydrogen water produced by the machine is such a free radical scavenger.

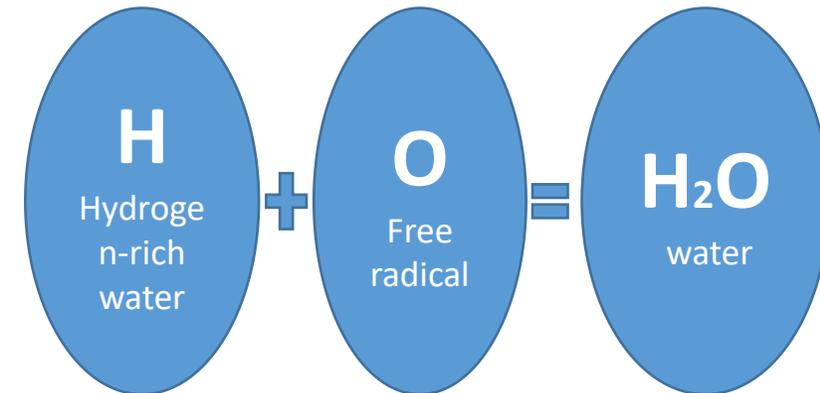

Hydration-Hydrogen Power Skin Beauty Machine Technology

- 1. Titanium Rapid Hydrogen Production Technology:** It can instantly produce hydrogen-rich water with a concentration of up to 1480 ppb and remove free radicals inside and outside cells.
- 2. Hydrogen Oxygen Separation Technology:** While generating hydrogen ion water, the effect of separating oxygen-enriched water to remove free radicals from the environment is discussed.
- 3. Smart sensor control:** The whole body is controlled by 36 sensors, intelligent management, safe, simple, efficient and anti-counterfeiting.
- 4. Two-way eddy current technology:** high-speed rotating vortex carries functional hydrogen water, which quickly penetrates inside and outside the cell to target free radical scavenging.

Hydration=Hydrogen Power Skin Beauty Machine

Hydrogen-rich ion water effect



Clean: For environmental radicals, remove PM2.5, dust, aphids, bacteria, and exhaust.

Detoxification: Blocking the attack of free radicals on cell membrane, nucleus and DNA, and detoxification inside and outside the cell.

Repair: repair the epidermal defense function, purify the keratosis caused by free radicals, fibrotic cells.


Maintenance: Maintain the dermis layer to prevent free radical degradation of collagen, fiber bundles, hyaluronic acid.

Hydration-Hydrogen Power Skin Beauty Machine Machine Layout

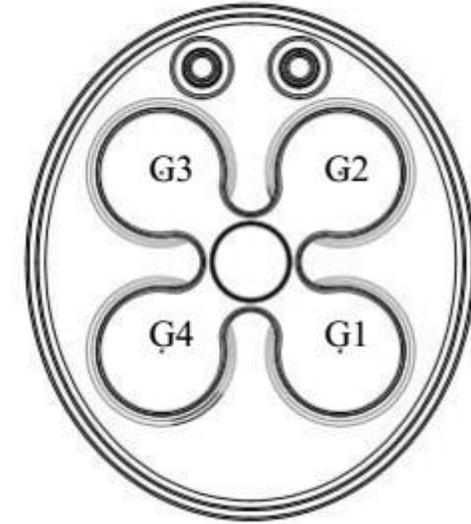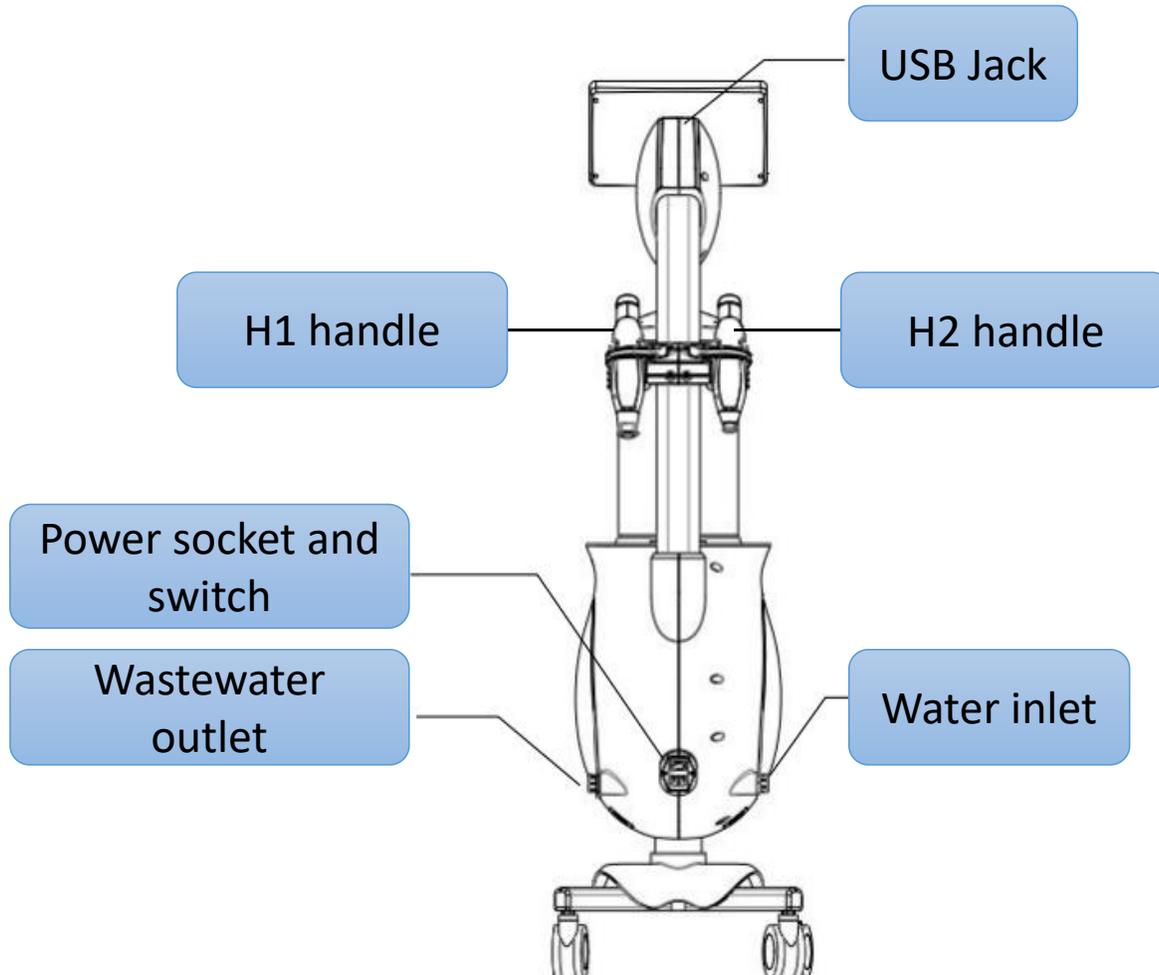

G1 G2 G3 G4: Four bottles of water in the corresponding location

Hydration-Hydrogen Power Skin Beauty Machine Operation Interface

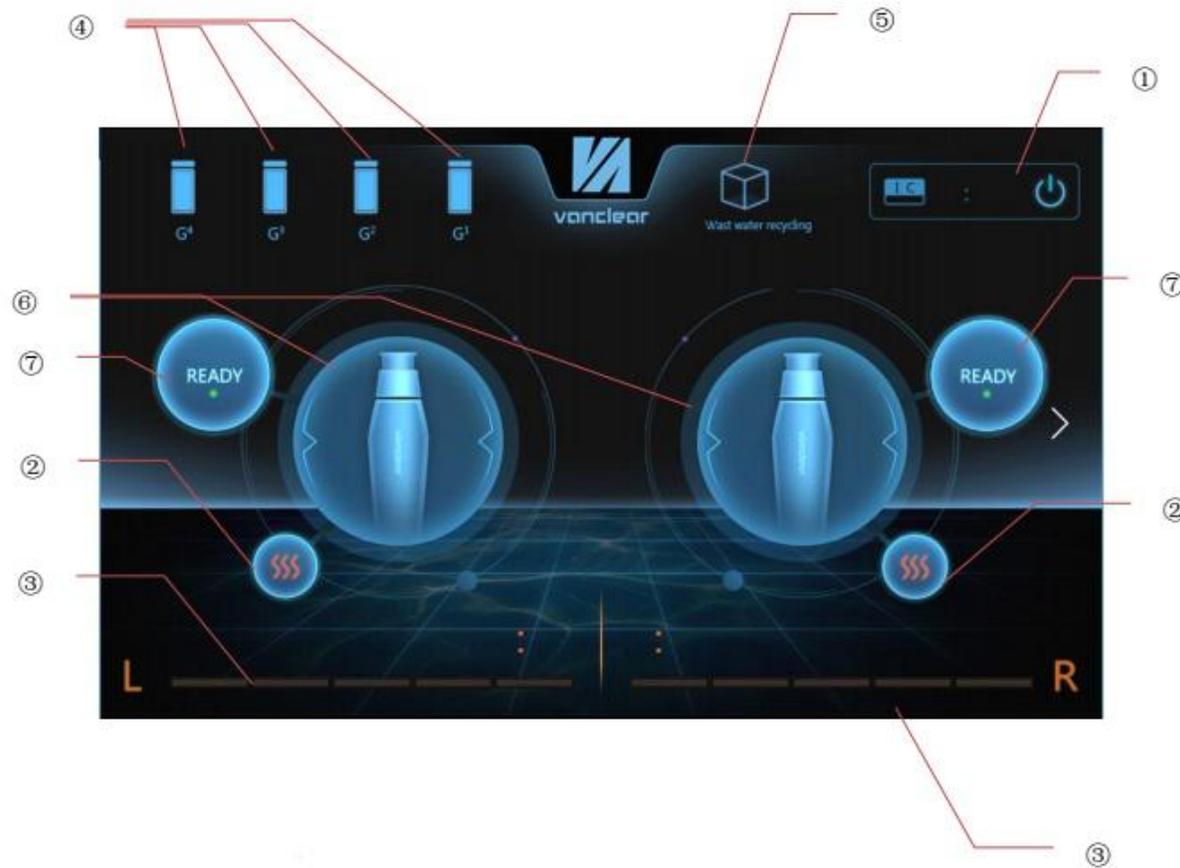

- ① IC card remaining time display
- ② Temporarily useless
- ③ Shows the remaining time for a single operation of the left and right handles
- ④ Choose G1-G4 water bottle and add water
- ⑤ Click this button to enter the waste water tank for drainage
- ⑥ Select to use left or right handle
- ⑦ Confirm whether to start working

Hydration-Hydrogen Power Skin Beauty Machine

General process

- Step 1 Install filter and apron



- Step 2 Install bottle and check it for leaks



- Step 3 Add liquid, cover it



- Step 4 Turn on the switch



Hydration-Hydrogen Power Skin Beauty Machine

General process

- Step 5 Insert the inlet pipe into the bucket
- Step 6 Click on the button to add water



- Step 7 Install the tips



- Step 8 Exclude wastewater after treatment



Hydration-Hydrogen Power Skin Beauty Machine

Handle operation steps: H1 handle

step 1 : The button turns purple then press it
With large round water head and G1 bottle



step 2 : Twist to the left, the button turns red,
then press it.
With small round water sculpture head and
G2 bottle

Efficacy: soften cuticle, remove surface dirt

Products used: pure water or softened
horny clean water

Operation time: 3 minutes

Cleaning sequence: cheeks - forehead (slip,
no pressurization)



Efficacy: remove pore dirt, blackheads and
whiteheads.

Products used: pure water or blackhead
whitehead function water

Operation time: 3 minutes

Cleaning sequence: nose-T area (dot type,
pressurization as needed)

Hydration-Hydrogen Power Skin Beauty Machine

Handle operation steps: H2 handle

step 3 : The button turns purple then press it
Injecting oxygen-enriched ion water with triangular head and G3 bottle



Efficacy: anti-inflammatory, bactericidal, calm skin, remove PM2.5, dust, mites, bacteria, exhaust.

Products used: pure water

Operating time: 6 minutes

Cleaning sequence: cheeks - forehead
(positioning point press type)



step 4 : Twist to the left, the button turns red, then press the it.
Injecting hydrogen-enriched ion water with triangular heads and G4 bottle



Efficacy: Remove free radicals inside and outside cells, reduce free radicals into water, block free radicals from attacking cell membranes, cell nucleus and DNA, and detoxify cells inside and outside the cell.

Products used: pure water

Operating time: 6 minutes

Cleaning sequence: cheeks - forehead
(positioning point press type)

Hydration=Hydrogen Power Skin Beauty Machine For people:

1. The stratum corneum is vulnerable to peeling, red blood and susceptible skin;
2. Large pores, oily skin, repeated blemish skin;
3. Drying, elastic drop;
4. Uneven skin color, dull, yellow, dull;
5. Long-term outdoor, air-conditioned environment, irregular life caused by loose skin, decreased thickness.



Hydration-Hydrogen Power Skin Beauty Machine

Precautions

1. Inflammation of the skin, severe allergies, damage, after the sun, during pregnancy
2. Avoid filling positions within 3 months of micro-filling
3. The skin is too thin, sensitive and sensitive
4. Heart bypass, three-high people do not operate the instrument
5. There is no special taboo on the diet
6. More water, pay attention to sun protection



